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Our company is producing hydrophilic catheter in Beyşehir District of Konya Province. Since it started to be a active producer, hight importance attached to the quality has always been the principle of our company. Production of urinary catheter is the subject of production of our company. SGD Medical is a company that has ISO 13485 and CE certificates. It is the first and the only company that makes production in the field of hydrophilic catheter production as an integrated facility. The gel on our medical equipment that make slipperiness easy, decreases friction by 85-95 percent. As a result of this, our product can be used easily without having pain.

It is produced to enable you to live in a more free and hygenic environment completely. Therefore, it will provide you the comfort to be more untroubled and secure in travel, shopping, school. The most important characteristics of our product is that it is used only once. Its distribution is made over all regions of the country. Works on exportation of the product continues. Our company will continue protect quality with its new products. Our company for which human health always comes first has been the proud of the region with the employment it has created.

Do not forget, we work for your health...

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We Care About Your Life Quality...
SGD (Tak) Catheter For
Intermittent Categorization

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Çapraz iki yan delik - Two transverse side holes

Renk kodlu funnel konnektör - Funnel connector with color code

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UTILIZATION OF HYDROPHILIC SOUNDER WITH GEL

Since mounting and detaching the Hydrophilic Sounder with Gel is easy, it is suitable for your urethra (Easy to Use). This catheter (sounder).The production is slippery while it is wet. When you use Hydrophilic Sounder with Gel, the only surface that is in contact with the sensitive urethra mucous is the water surface. You will not be bothered by unpleasant friction and the lubricant utilization thanks to Hydrophilic Sounder with Gel. Before using, submerge the Hydrophilic Sounder with Gel into water only for 30 seconds. So, the catheter will connect the water to the surface and it will enable the surface remains wet during the catheterization. This way, it will be easy for you while mounting and detaching the Hydrophilic Sounder with Gel.

Boyut Dimension	Renk Colour	Çap Diameter
06 ch	Ağır Yeşil Light Green	○
08 ch	Mavi Blue	○
10 ch	Siyah Black	○
12 ch	Beyaz White	○
14 ch	Yeşil Green	○
16 ch	Turuncu Orange	○
18 ch	Kırmızı Red	○
20 ch	Sarı Yellow	○
24 ch	Koyu Mavi Dark Blue	○

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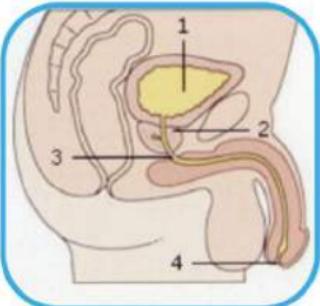
UTILIZATION OF HYDROPHILIC SOUNDER WITH GEL FOR MALES

Male sub - urinary system section. Male urethra is 15 -18 cm of length and in the shape of S.

By way of pulling the penis towards the abdominal, it enables that the S shape is corrected and putting the catheter becomes easy.

- 1- Wash your hands with plenty of water and prepare the catheter.
- 2- Find the most suitable position for yourself. You can sit or stand up, as long as you are comfortable.
- 3- Pull the penis towards the abdominal. Likewise, your urethra shall be corrected and it shall be easier for you to put the catheter.
- 4- Do not tighten your penis too much.
- 5- Slightly twist the catheter toward the front before putting it. Therefore urine does not drain away.
- 6- Put the catheter slightly towards the bladder by the way of urethra until urine comes.
- 7- When you have urinated, pull your penis towards the abdominal again and remove the catheter.
- 8- Throw the catheter away and wash your hands.

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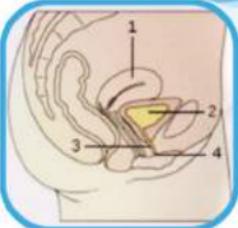
UTILIZATION OF HYDROPHILIC SOUNDER WITH GEL FOR FEMALES

The urethra aperture is like a small star. In the beginning, a mirror can help you to find the right location. After you get used to mounting the catheter, to find the urethra aperture by means of feeling is much easier and it shall enable that you are more flexible.

- 1- Wash your hands with plenty of water and prepare the catheter.
- 2- Find a suitable position for yourself. Try to seat in a comfortable way.
- 3- At the beginning, a mirror will help you to find your urethra aperture.
- 4- Push your thigh slightly towards the front.
- 5- Split the lips with the help of your fingers and pull it slightly upwards. So that you can see the urethra aperture much easily.
- 6- Slightly twist the catheter towards the front before mounting it. So urine does not drain away.
- 7- Put the catheter slightly towards the bladder by the way of urethra until urine comes.
- 8- After you have urinated, remove the catheter.
- 9- Throw the catheter away and wash your hands.

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İmalat İhr. İth. Ltd. Şti.